



Shoulder Immobilizer - Instructions

Please read carefully.

THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER MEDICAL AUTHORITY. PROPER FIT AND APPLICATION OF THIS PRODUCT MUST BE PERFORMED BY A MEDICAL PROFESSIONAL. THESE DIRECTIONS ARE GUIDELINES ONLY AND ARE NOT OFFERED AS MEDICAL RECOMMENDATIONS.

Your doctor or orthotist may alter some of these general instructions. Your doctor and orthotist know your medical status and are using this orthosis as part of your total healthcare. It is important to follow their instructions carefully, even if they differ from some of the following guidelines.

***APPLYING THE ORTHOSIS** Your orthosis original fitting must be done by a qualified health care practitioner. The orthosis should already be adjusted to fit your body*

Applying Shoulder Immobilizers:

Wrap body band around torso. Apply arm cuff directly under arm and secure around bicep. Apply shoulder strap to posterior section of body band. Adjust shoulder strap over shoulder to comfortable position at wrist. Fasten anterior shoulder strap with patient's arm at comfortable angle.

Washing Instructions:

Hand wash support in mild soapy cool water. Rinse thoroughly and Drip Dry on flat surface.

Do Not use solvents or chlorine bleach.

Do Not put in the dryer.

Do Not iron.

Note:

Always consult with a Health Care Practitioner before making any changes in your support.

Discontinue use at once if any sign of dermatitis or other problem occurs. If discomfort occurs while wearing support, discontinue use at once and contact fitter or physician. Discontinue use at once if signs of prolonged wear affect usefulness of support. Contact fitter or physician with any questions about proper care or use. This item is intended for use in medical treatment, and not to be worn as a protective device during sports activity. Support Should Not be worn while sleeping. Support Should Not be worn by persons with phlebitis or circulatory problems. Never wear support over an open wound.

