



Abdominal Binder

THIS PRODUCT INTENDED FOR THE APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER MEDICAL AUTHORITY. PROPER FIT AND APPLICATION OF THIS PRODUCT MUST BE PERFORMED BY A MEDICAL PROFESSIONAL. THESE DIRECTIONS ARE GUIDELINES ONLY AND ARE NOT OFFERED AS MEDICAL RECOMMENDATIONS.

Your doctor or orthotist may alter some of these general instructions. Your doctor and orthotist know your medical status and are using this orthosis as part of your treatment plan. It is important to follow their instructions carefully, even if they differ from some of the following guidelines.

Applying binders:

Center the support on your back. Pull left side across your middle section and then pull the right side over the top and attach the Velcro closure.

To Remove: Unfasten the Velcro closures and reapply neatly to flat garment.

Washing Instructions:

Hand wash support in mild soapy cool water.

Rinse thoroughly and drip dry on a flat surface.

Do Not use solvents or chlorine bleach.

Do Not put in the dryer.

Do Not iron.

Do Not wear directly against the skin. An undershirt or similar article of clothing should be worn between the support and your body.

Do Not wear if you have broken skin or a known susceptibility to dermatitis. If a rash or skin irritation develops, discontinue use immediately and consult a physician.

Do Not wear while sleeping.

Do Not wear if you have phlebitis or circulatory problems.

Never wear support over an open wound.

Notice

While every effort has been made to obtain maximum compatibility of function, strength, durability and comfort, this device is only one element in the overall treatment program administered by a medical professional.

U.S. Orthotics disclaims any warranty, expressed or implied, that these products prevent injuries.

